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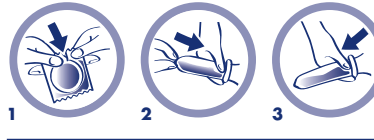
EN Instruction for use of Ritex condoms

Tear open the individual container **carefully** from the **serrated edge**. Please note that for example fingernails, jewellery or piercings of the private parts can damage the condom **(1)**. Before there is any contact between the penis and your partners body, **pull the foreskin back completely**, place the condom with **the roll on the outside** on the glans of the erect penis, squeeze the test using your thumb and forefinger and roll the condom all the way down **(2)**. In case the penis begins to limp or if you feel that the condom might slip off or if it tightens excessively onto the penis (danger of breakage), withdraw the penis while holding the condom firmly in place at the base of the penis (1). Seek medical assistance as soon as possible, at least within 72 h. Do not use a condom late, burst during use. Ritex condoms are medical devices intended for the use in vaginal intercourse as a method of contraception and localised protection against sexually transmitted infection. Non-vaginal use or low levels of vaginal moisture jeopardise the safety of condoms. No method of contraception can guarantee 100% protection. In order to remedy a **lack of slide**, for example with insufficient vaginal dryness or anal intercourse, additional condom-compatible **Ritex lubricants** such as GEL<sup>®</sup>, HYDRO, BIO or LONGTIME should be used in sufficient quantities on the **outer side of the condom**. Never use additional lubricants directly on the penis or the inside of the condom, as this increases the risk of the condom slipping or tearing. Never use lubricants containing oil or grease (e.g. vaseline, baby oil, body lotion). Only use those which are recommended by the manufacturer for use with latex condoms. Consult your doctor or pharmacist to determine whether the use of pharmaceuticals such as suppositories, ointments, creams etc. on either vagina or penis may impair the safety of the condom. Store at room temperature in the original sealed individual container only. Only dispose of used condoms via the household waste. Do not flush them down the toilet.

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